SANISAN HOME ®

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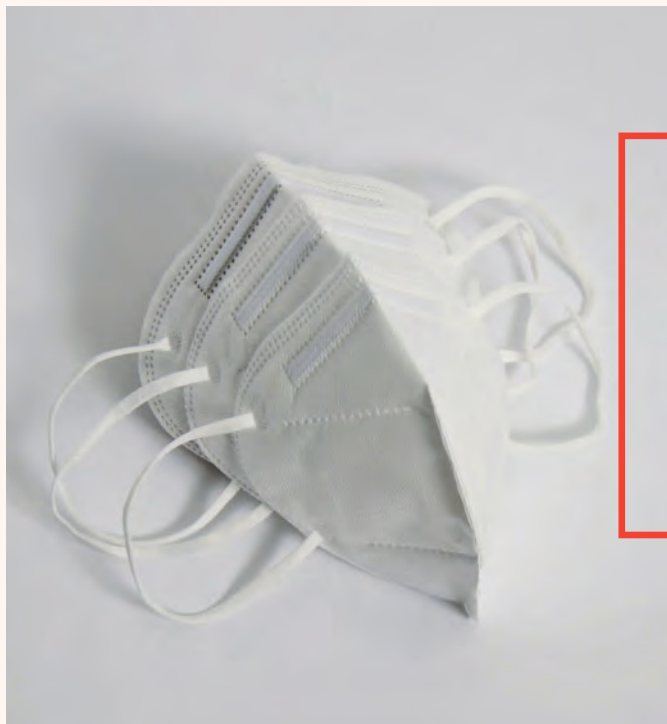
AUSTRIA

FFP2 Respirator Mask

1. MADE IN AUSTRIA
2. Self and third party protection
3. Great comfort
4. High breathability due to high quality non-woven materials

1. Wash your hands using soap and water or a hand sanitizer.

2. Pull the loops of the mask back to fit them around your ears.

3. Pull the upper and lower ends of the mask over nose and chin, so that no open gaps remain between face and mask.

4. Lightly press the metal nose piece in your mask across nose and cheeks to fit.

5. Avoid touching the mask while wearing.

6. Dispose of the mask safely immediately after wear.

Mask Specifications


Brand	Hygiene Austria LP GmbH
Type	FFP2 Respirator Mask
Quantity/Units	Packs of 3 pc., 20 pc.
Colour	White
Expiration Date	5 years
Instructions	<ul style="list-style-type: none">- Suitable for personal safety and healthcare professionals- It is recommended to remove and dispose of the mask every 4 hours or as soon as humid.- Not reusable
Storage Conditions	Store above 0°C and below 40°C in a dry place without direct sunlight
Size	16x11cm folded (slight deviations possible)
Weight	5.25 g (slight deviations possible)
Certificates	CPA-FFP2 OETI certification in accordance with GZ 2020-0.247.451
Type	PSA-Category III
Device Category	FFP2
Product specifications	<ul style="list-style-type: none">- Made of 5-layer non-woven material, 2 outer layers made of spunbond fleece, 2 inner layers made of hot-air-cotton and 1 inner layer made of meltblown fleece polypropylene- 9cm extra strong and flexible nose piece made of two metal wires coated in rubber

Please check the authenticity of all certificates offered to you by suppliers through:
<https://www.eu-esf.org/covid-19/4513-covid-19-suspicious-certificates-for-ppe>
Your health will thank you.


Packaging

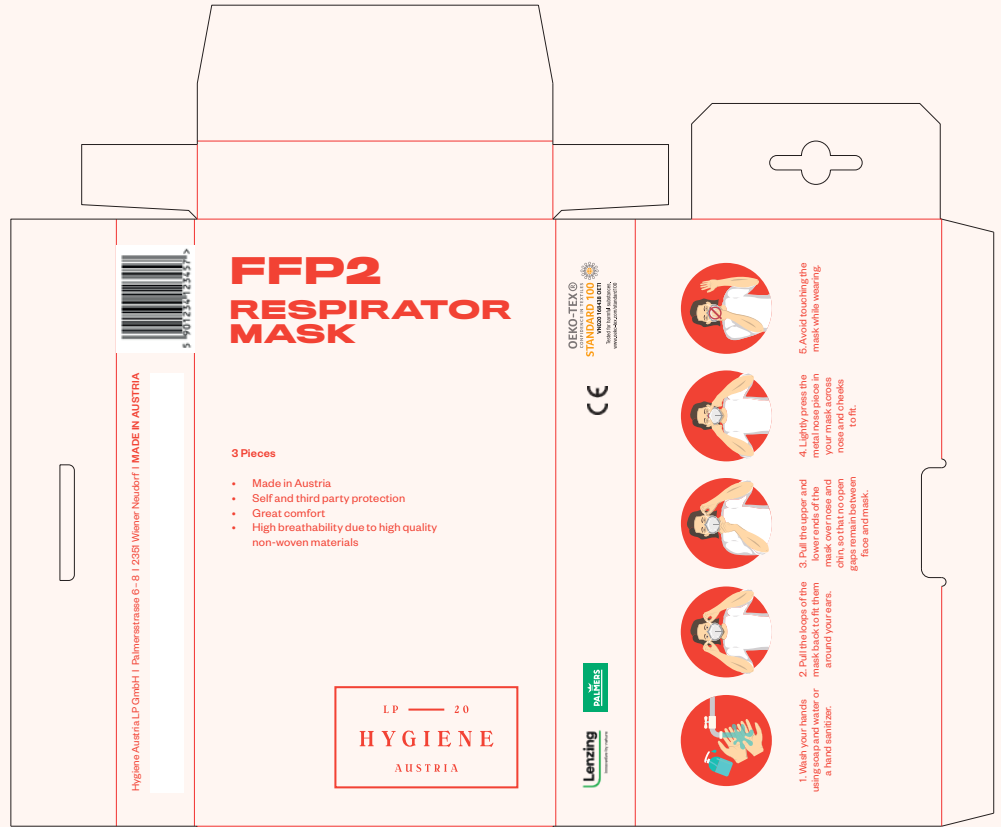Outside

Inside

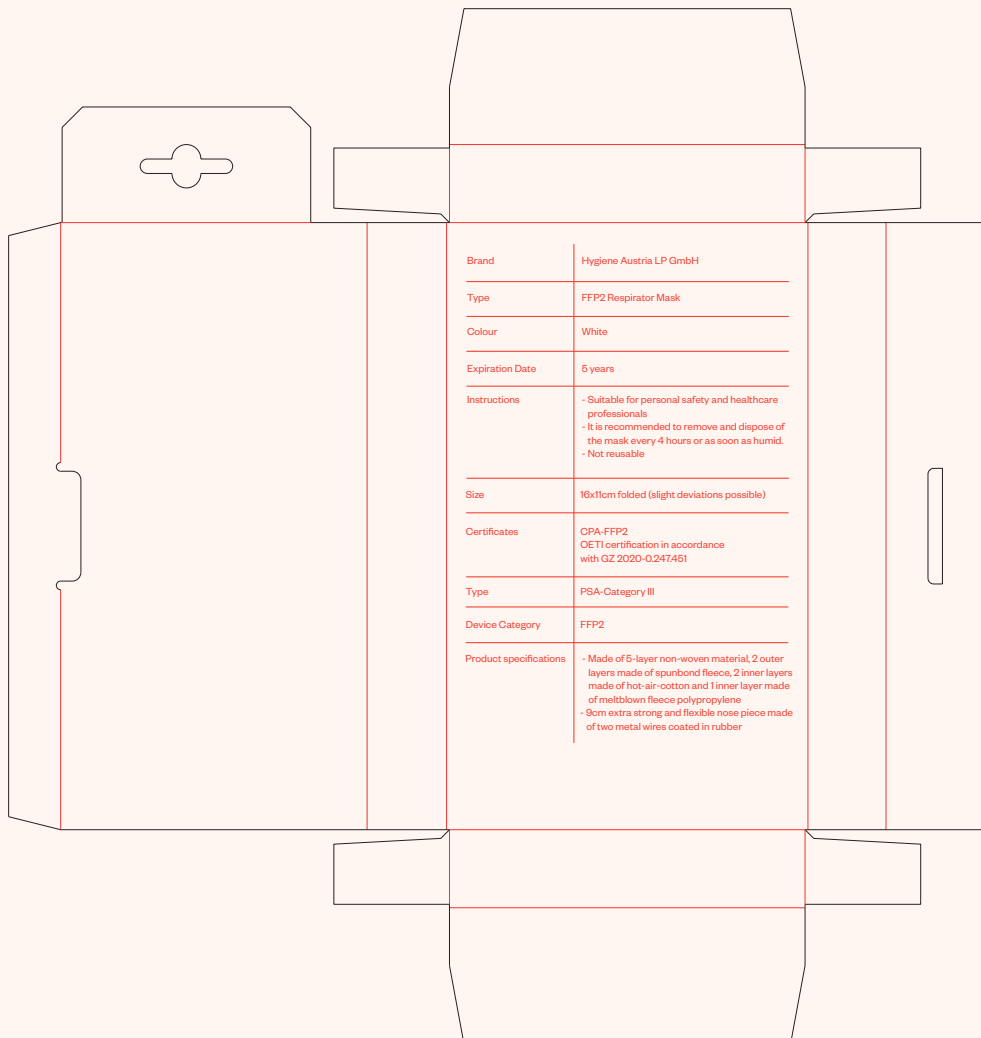Certificate to Letter of assessment VN635 170132

in accordance with enactment GZ 2020-0.247.451 as well as according to "Test principle for Corona SARS-Cov-2 pandemic respirators Rev. 0 dated 20.03.2020" for placing on the market in Austria

(Order VN635 170132.1 / Test-Report PTP-Test-Report No.: T20-0877(dated 2020-06-04))

Issued to distributor

Hygiene Austria LP GmbH
Palmerstraße 6-8
2351 Wiener Neudorf
AUSTRIA

manufacturer

Hygiene Austria LP GmbH
Palmerstraße 6-8
2351 Wiener Neudorf
AUSTRIA

Based on the documents submitted and the examinations carried out this is to certify, that the Corona Virus Pandemic Respirator (CPA)

"Atemschutzmaske, FFP2"

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(photo documentation see reverse side)

corresponds to the requirements of the shortened assessment process in accordance with enactment GZ 2020-0.247.451 as well as according to "Test principle") for Corona SARS-Cov-2 pandemic respirators Rev. 0 dated 20.03.2020" for placing on the market in Austria based on EN 149:2001+A1:2009, the following can be stated:

The Corona Virus Pandemic Respirator (CPA) passed the test with a positive result ^{note)}

^{note)} The fit of the mask depends on the face shape of the wearer, so a fit check must be carried out before use. Appropriate information must be enclosed with the mask.

This new PPE will be assigned to **category III**

This letter of assessment applies only to Corona Virus Pandemic Respirator (CPA) to medical professionals for the duration of the current health threat and / or until the GZ 2020-0.247.451 decree expires.

Ing. Judith Pointner
Head of Certification Body

OETI - Institute for Ecology, Technology and Innovation; Notified Body No. NB 0534
Vienna, 15.06.2020

*) non-harmonized standard

"Certificate" Page 1 of 2 Pages
certificate to Letter of assessment VN635 170132

CERTIFICATE

photo documentation